



User Manual



ALT-R008

Four-Wheel Aluminium
2-in-1 Wheelchair Rollator



UK Version

The **Alerta Four-Wheel Aluminium 2-in-1 Wheelchair Rollator** is an excellent dual purpose, hybrid mobility companion.

Its adaptability enables use as a sturdy lightweight rollator promoting independence, exercise and well-being, but also easily converts into a wheelchair by simply swinging the footrests into place and flipping the backrest over. It is the perfect friend promoting mobility by enabling the user to walk as far as they wish to without restricting distance from home, because it can then become a wheelchair and the journey can continue with the help of an assistant.

Alerta's Wheelchair Rollator surrounds the user with luxurious comforts such as soft extra-comfort padded seat, generously sized sturdy comfort backrest, quick release adjustable height ergonomic shaped handles, pull up break & push down hand-brake lock, easy swivel shock-absorbing wheels, storage bag with zip, swing-away detachable footrests, fixed padded armrests for wheelchair position, and easy one-click pull rapid fold for storage and transport.

Rollator Assembly

1. Carefully remove all packaging
2. Remove the rollator from the box
3. Unfold the rollator and push down firmly on the 2 outer seat poles mounts to secure into an open position with a clear audible click
4. Insert the 2 front wheels into the rollator main frame and secure with tightening knob (1)
5. Insert the 2 rear wheels on their frame mountings into the rollator main frame and click into position (2) and secure with tightening knob
6. Attach the shopping/storage bag to the front of the frame (3)
7. Place the footrests onto the frame by aligning the pin to the holes (3)



Operation

Brakes

1. To stop, squeeze the brake handle upwards. (4) Release hand brakes to walk.
2. To lock brakes, push the brake handle downwards (5)
3. To unlock brakes, lift hand brake.

Operation

Folding

4. To fold or store the rollator, turn footrest plates vertical, (you may remove the footrests completely if easier) and now pull up on the handle (6) in the centre of the seat and the rollator will fold together.

Height Adjustment


5. To adjust the height, press the release button on side frame (7) below the handle and adjust the handle to desired height, then repeat for the other side. Make sure they are securely locked and adjusted to the same height for both sides.

Footrests

6. The footrest plates can be folded up and down, (8) or folded right back out the way and clipped open (9) for use when walking, or removed totally if desired. (10)

Reversible Backrest

7. The backrest faces forward (11) when walking forward as a rollator, or is flipped over and faces backwards (12) when using as a wheelchair.

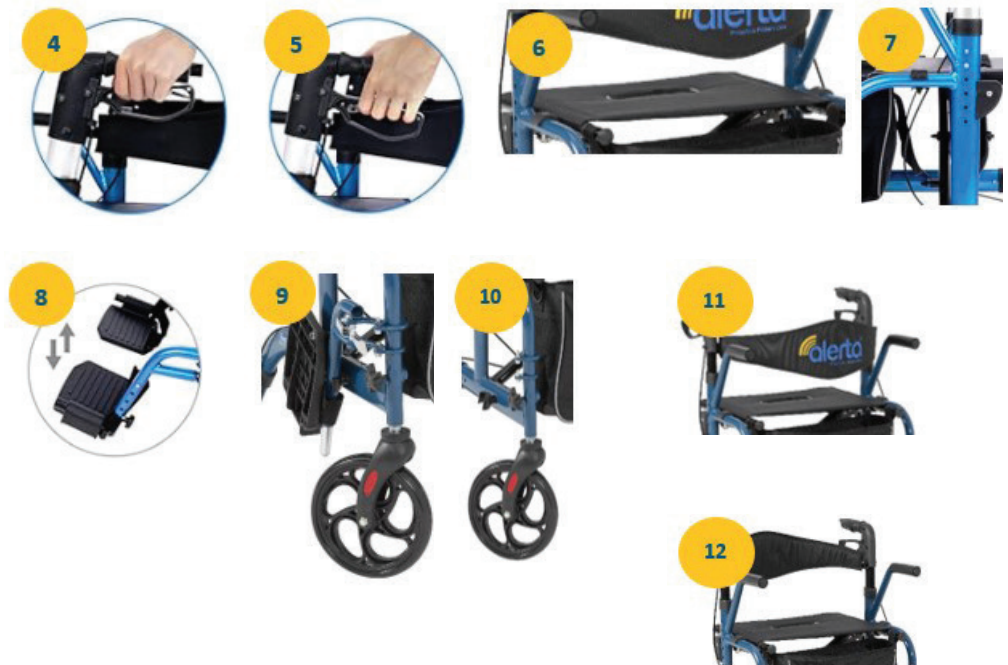

Maintenance

1. You should clean/disinfect the Wheelchair Rollator at regular intervals (mandatory at the time of user change)
2. For dust and dirt, wipe rollator with a damp cloth
3. Use mild soap and water to clean. Dry with soft cloth
4. Never use abrasive or corrosive cleaning agents or high-pressure and steam cleaners
5. Check fit and tightness of knobs, nuts and bolts on a weekly basis to ensure Wheelchair Rollator is safe to use
6. Make sure that no dirt is deposited on the tyres / brakes. This can be removed with a soft brush and warm water

Main Components of Wheelchair Rollator



1. Ergonomic comfort handle
2. Brake handle
3. Handle height adjustment part
4. Storage bag with zip & shoulder strap
5. Front wheel
6. Rear wheel
7. Soft extra-comfort padded seat
8. Sturdy comfort backrest (flip for wheelchair)
9. Swing-away detachable footrests
10. Fixed padded armrests for wheelchair position
11. Walking stick stand

CAUTION

- Do not use this product without reading this manual first
- This product has a patient weight limitation of 136kgs
- The storage bag can be loaded up to a max. 5 kg and is intended solely for the transport of goods
- Test the brakes before using and check the brakes after each height adjustment
- Do not hang any objects on the handle or brake handle
- Before each use, check whether the Wheelchair Rollator is completely unfolded and whether the handles are at the same height.
- Alerta Medical assumes no liability for any damage or injury caused by improper use of this product
- If components are missing or damaged, DO NOT USE. Contact your dealer immediately
- Ensure all parts and components are tightened before use
- Use with caution when operating Wheelchair Rollator on wet or slippery surfaces. Do not use on steep inclines or stairs
- Inspect Wheelchair Rollator weekly for proper function and wear. If any component is not working or is worn, please contact your dealer immediately for replacement parts
- Special users, such as people with poor psychological development, need to be assisted by professional medical supervisor
- If any serious incident occurs with the device it should be reported to the manufacturer and local authorized dealer



CS Lifesciences Europe Ltd

The Black Church
St. Mary's Place
Dublin 7
D07 P4AX
Ireland

eurep@cslifesciences.com

Rev: 2025.09

Making **cutting-edge** medical equipment **affordable**

Head Office

Alerta Medical
4 Symington Place
Riverside Business Park
Irvine
KA11 5DE
United Kingdom

www.alertamedical.com

